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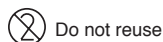
【Contraindications and Prohibitions】

<Applications (Applicable Patients)>

Do not use the product on any patients who are sensitive or allergic to metal.

<Prohibitions>

Re-use prohibited

Re-sterilize prohibited

Do not use the product when the aseptic condition of the products is considered to be lost due to damage/breakage of package.

Do not perform any second processing such as disassembling or modifying.

【Shape and Structure】

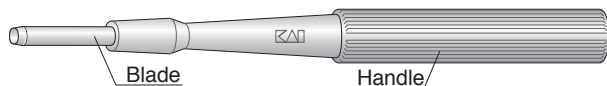
This product is a disposable biopsy punch which consists of blade and handle (Plunger and spring are also included as the part for the plunger type).

1. Physical Appearance


Regular type (Blade length 7mm)

Long type (Blade length 15mm)

Plunger type

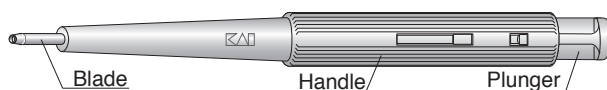2. Material

Blade: Stainless Steel (Nickel and Chromium contained)

【Purpose】

This product is a device for perforating the subcutaneous of human body and sampling the tissue for examination, treatment and diagnostic. This product is single use.

【Handling and Use】

① Before Use

Confirm there is no damage on the sterilized pack, open the package and remove the product.

Confirm there is no damage/breakage of the product and use it without intermission.

② Handling and Use

Push the blade horizontally against the subject tissue. Rotate the product and increasingly apply a load on it, then cut the tissue off.

③ Handling and Use for the Product with the Plunger

After cutting the tissue off, push the plunger and extrude the tissue which is remained inside the blade.

※ If the literature on the skin biopsy technique with the product is in need, please contact the distributor.

【Precautions for Use】

1. Important Precautions

- (1) Do not use the product if the package is damaged/broken (The aseptic condition cannot be assured even though the product is within their expiry date).
- (2) When removing the product from the package, do this procedure with care of the blade. If the blade gets contact with anything else, do not use the product and replace it with a new one.
- (3) Do not use the product if any abnormality such as damage/breakage is recognized.
- (4) Handle the products with care in order to avoid being injured by the blades (The blade is so sharp that it would injure).
- (5) When perforating the skin tissue, be aware of the skin thickness and handle the product carefully in order not to damage the other unnecessary skin tissue.
- (6) After the usage of the products, be aware of the risk of infections and dispose them properly as medical waste with the perspective of preventing infection.

2. Error・Adverse Event

- (1) Error
 - Unintended perforating
 - Puncture failure
- (2) Adverse Event
 - Deep tissue injury (a tendon etc.)

【Storage and Expiration Date】

1. Storage

- (1) Avoid exposing the product to high temperature, high humidity, direct sunlight and water. Store the product at the ambient temperature.
- (2) Store the product in a clean and dry place without loading anything on the products.
- (3) Do not store the product together with chemical products or in an area where gas occurs.
- (4) Avoid damaging or making pinholes to package materials.

2. Expiration Date

Check the expiry date which is indicated on the individual and external package.

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Kai Industries Co., Ltd.
1110, Oyana, Seki City, Gifu Prefecture,
501-3992, Japan

【EU Authorized Representative】

Kai Europe GmbH
Kottendorfer Straße 5
42697 Solingen, Germany